




QUICK START GUIDE

CardioTech GT-175 EKG Machine

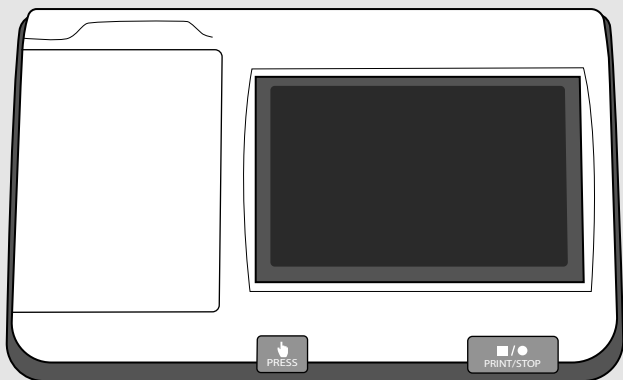



CardioTech GT-175

Contact Us:

info@cardiacdirect.com
888-354-2968

Contents

02

Basic Setup

- 02 Installing The Battery
- 03 Powering On The CardioTech GT-175
- 03 Connecting The Patient Cable To The EKG Machine
- 04 Loading Paper Into The Printer
- 05 Settings Change To Print Interpretation

07



FAQ

- 07 "Lead Off" Error


Basic Setup

Installing The Battery

1. Gently flip the CardioTech GT-175 upside down (screen side down) to reveal the underside of the machine.
2. Use a screwdriver to unscrew the battery compartment cover off. **(Figure A)**
3. Insert the battery wire into the socket. Then, place the battery in the compartment. **(Figure B)**

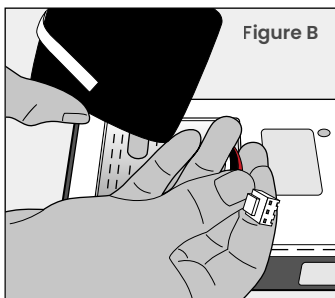
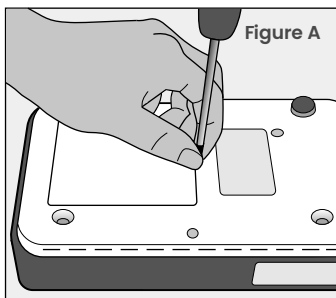


4. Ensure that the white tab is at the top. **(Figure C)**
5. Replace the battery compartment cover, and revert the machine back to its original position.

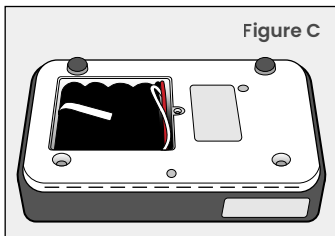


Powering On The CardioTech GT-175

1. Plug one end of the AC Power Cable into the power supply socket, located on the backside of the GT-175. Plug the other end into an AC Wall Outlet. **(Figure D)**
2. Press and hold the power button on the right side of the GT-175 to power on the machine. **(Figure E)**

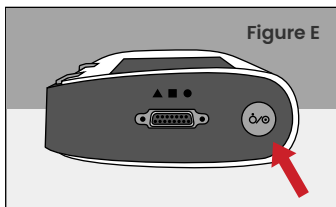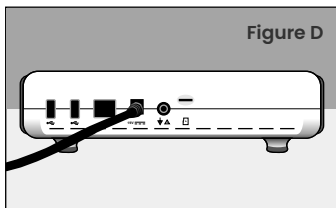



Connecting The Patient Cable To The EKG Machine

1. Connect the patient cable to the patient cable socket on the right side of the main unit, and then secure them with two screws. **(Figure F)**

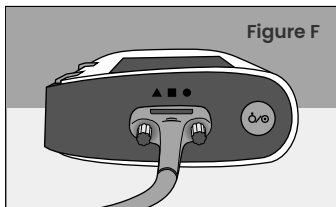


Loading Paper Into The Printer

1. Push the 'Paper Replacement' button to open the printer door. Lift the door all the way open. **(Figure G)**
2. Put the roller through the paper roll, and insert into the paper compartment. **(Figure H)**
3. Before placing the paper into the printer, orient the paper so that the black square is in the bottom left corner.

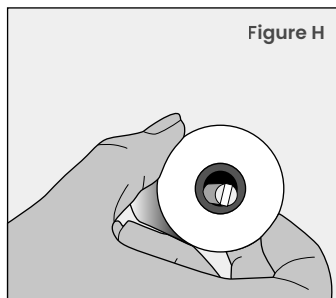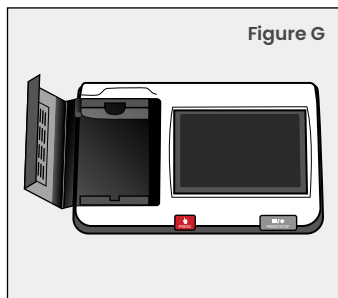

4. Close the printer door, with the first page of paper hanging over the printer. Tear the first page, and the printer is ready to be used. **(Figure I)**

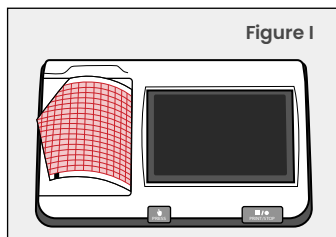

Settings Change To Print Interpretation

1. In system setup, click 'record info'.
2. Select 'paper marker'.
3. Switch to 'no'.
4. The GT-175 is ready to print interpretations.

BASIC SETUP COMPLETE

*Continue to Next Pages for
FAQ & Troubleshoot Issues*

FAQ

“Lead Off” Error

The leads are not connected well. Please check whether the electrodes are connected to the patient skin well, and then make sure that the patient cable socket is connected to the patient cable firmly.



*Scan here for additional
assistance setting up
your product*



*Providing Value
Without Sacrifice*

5963 Olivas Park Drive
Suite F, Ventura
CA 93003

info@cardiacdirect.com
888-354-2968